**NSK**

OPERATION  
MANUAL

Contra Angle Handpiece

**FX15 / FX15m**

**FX23 / FX23m**

**FX25 / FX25m**

**FX75 / FX75m**


FX15 / FX15m / FX23 / FX23m

FX25 / FX25m / FX75 / FX75m


CE 0197

MADE IN JAPAN


OM-C0438E 001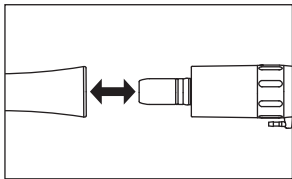


1

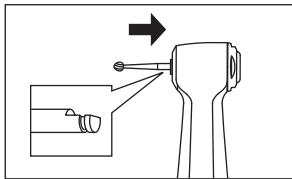
2

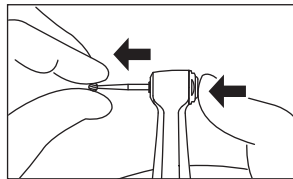
3

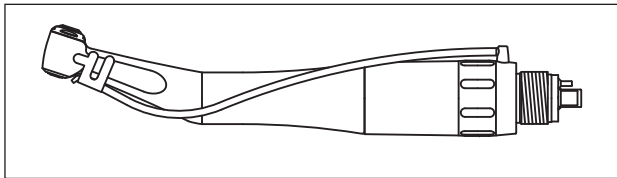
4

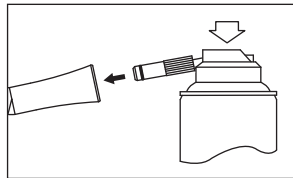
5

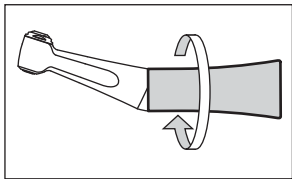
6

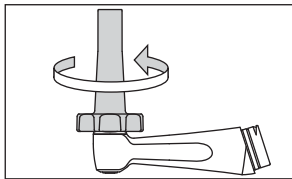7

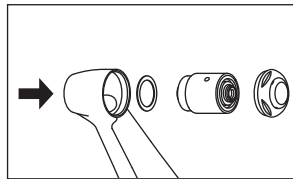8

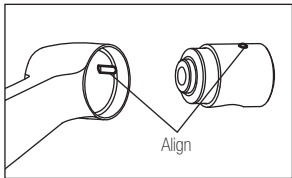9

## 1. User and Intended Use

User : Qualified Professionals

Intended Use : Dental Treatment

(FX15/FX15m/FX23/FX23m/FX25/FX25m)

Dental Treatment (Endodontics) (FX75/FX75m)

## 2. Precautions for handling and operation

- Please read these precautions carefully and use only as intended or instructed.
- Safety instructions are intended to avoid potential hazards that could result in personal injury or damage to the device. Safety instructions are classified as follows in accordance with the seriousness of the risk.

Class	Degree of Risk
 <b>CAUTION</b>	Hazard that could result in light or moderate injury or damage to the device if the safety instructions are not correctly followed.
<b>NOTICE</b>	General product specification information highlighted to avoid product malfunction and performance reduction.

### CAUTION

- Read this Operation Manual before use to fully understand the product functions and file for future reference.
- When operating the product always consider the safety of the patient.
- This product does not consider patient's age (except infants), gender, weight or nationality.
- This product does not consider operator's age (mature person), height, weight, gender, or nationality.
- Users are responsible for the operational control, maintenance and continual inspection of this product.
- Do not attempt to disassemble the product nor tamper with the mechanism except as recommend by NSK in this Operation Manual.
- Do not allow any impact on to the product. Do not drop the product.
- Operators and all others in the area must wear eye protection and a mask when operating this handpiece.
- Should the product function abnormally, cease operation

immediately and contact your Authorized NSK Dealer.

- Depressing the Push Button while the handpiece is in operation may cause overheating, serious technical damage and possible premature handpiece failure. During operation avoid contact with any oral tissue that may cause the Push Button to be depressed while the handpiece is in operation.
- Do not use high acid water or sterilizing solutions to wipe, immerse or clean the handpiece.
- The products are delivered in a non-sterile condition and must be autoclaved prior to use.
- Perform regular function and maintenance checks.
- If the product is not used for a long period check it is functioning correctly before using on a patient.
- To avoid clinical downtime it is recommended that a spare be kept on hand in case of a breakdown during surgery.
- U.S. Federal law restricts this device to sale by or on the order of a licensed physician.

### 3. Connection & Disconnection of the Handpiece and Motor

#### 3-1 Connection

- 1) Connect the handpiece direct to the motor. (Fig.1)
- 2) Make sure the handpiece is firmly connected to the motor.

#### 3-2 Disconnection

Hold the motor and the handpiece separately and disconnect with care.

#### CAUTION

- Disconnect the handpiece only after the motor has completely stopped rotating.
- Connect ONLY to E type motors (ISO 3964).

### 4. Insertion & Removal of the Bur/File

#### 4-1 To Insert the Bur (FX15/FX15m/FX23/FX23m/FX25/FX25m)

- 1) Insert the bur until it is correctly seated in place.
- 2) Depress the Push Button and insert the bur into the chuck until the

bur "notch" mechanism engages. Release the button. (Fig.2)

- 3) Ensure that the bur is secure by gently pulling and pushing the bur WITHOUT depressing the Push Button.

#### 4-2 To Insert the File (FX75/FX75m)

- 1) Insert the file until it is correctly seated in place.
- 2) Depress the Push Button and insert the file into the chuck until the file "notch" mechanism engages. Release the button.
- 3) Ensure that the file is secure by gently pulling and pushing the file WITHOUT depressing the Push Button.

#### 4-3 To Remove the Bur/File

Depress the Push Button firmly and remove the bur/file. (Fig.3)

### CAUTION

- Always insert the bur or file all the way into the chuck.
- Remove the bur or file only after the handpiece has completely stopped rotating.
- Always keep the bur or file shank clean. Entry of debris into the

chuck, via the bur shank, could cause bur rotation slip and also prevent the bur from being securely located in the chuck.

- Do not exceed the bur or file speed recommended by the bur or file manufacturer.
- Do not exceed the drive motor speed or torque recommended by the file manufacturer.
- Do not exceed maximum bur length recommended by the handpiece manufacturer.
- Do not apply excess pressure to the bur/file as it may break or bend or become difficult to remove.
- DO NOT use bur/file with problems listed below as the bur/file may break or disengage from the chuck.
  - Bent, deformed, worn, rusted, broken, deficient bur.
  - Bur which is cracked on the edge or axis.
  - Non-ISO standard, or tampered bur.

## 5. Connection of the Spray Nozzle (FX23/FX23m/FX25/FX25m)

- 1) Attach the Spray Nozzle (Optional) to the handpiece head.
- 2) Cut the Irrigation Tube to the desired length then connect to the Spray Tube Nozzle on the motor. (Fig.4)

## 6. Check before treatment

Check that the Head Cap is firmly tightened. Also check for handpiece vibration, noise and overheating. If any abnormalities are found do not use the handpiece and contact your Authorized NSK Dealer.

## 7. Maintenance

After each patient maintain the product as follows.

### 7-1 Cleaning (Handpiece)

- 1) Remove dirt and debris from the product. Do not use a wire brush.
- 2) Wipe clean with alcohol-immersed cotton swab or cloth.

This icon denotes that the product can be washed via Thermo-Disinfector. (FX15m/FX23m/FX25m/FX75m)

Refer to the Thermo-Disinfector manual.

### CAUTION

- After washing with Thermo-Disinfector and prior to lubrication, dry the product until all internal moisture is thoroughly removed. Thermo-Disinfector moisture remaining inside the product could reduce the effect of lubrication and could cause corrosion inside of the product.
- To clean the product never use any solvent such as benzine or thinner.

### 7-2 Lubrication

#### ■ NSK PANA SPRAY Plus

Apply NSK PANA SPRAY Plus every time after each use and/or before autoclaving.

- 1) Remove the bur or file from the handpiece.
- 2) Insert the Spray Nozzle into the Spray Port nozzle on the can.
- 3) Insert the Spray Nozzle in rear of the handpiece. Hold the handpiece and spray for approximately 2-3 seconds. Apply

lubricant until it expels from the handpiece head for at least 2 seconds. (Fig.5)

### CAUTION

- When applying spray be sure to hold the handpiece firmly to prevent the handpiece from slipping out of the hand due to the spray pressure.
- Hold the spray can upright.

#### ■ NSK Care3 Plus

When using NSK Care3 Plus automatic handpiece cleaning and lubrication system, refer to the Care3 System instructions.

### 7-3 Sterilization

Sterilize the product by autoclave sterilization. Remove the bur or file after each patient and sterilize as noted below.

- 1) Insert an autoclave pouch. Seal the pouch.
- 2) Autoclavable under the conditions below.

Autoclave for more than 20 min. at 121°C, or 15 min. at 132°C, or 3 min. at 134°C.

- 3) The handpiece should remain in the autoclave pouch until required for use.

### CAUTION

- Do not autoclave the product with other instruments even when it is in a pouch. This is to prevent possible discoloration and damage to the product from chemical residue on other instruments.
- Keep the product in suitable atmospheric pressure, temperature, humidity, ventilation, and sunlight. The air should be free from dust, salt and sulphur.
- Immediately after use, the handpiece should be cleaned, lubricated and sterilized. If blood remains on the external or internal surfaces it can become clotted and cause rust.
- Do not heat or cool the product too quickly. Rapid change in temperature could cause damage to the product.
- If the sterilizer chamber temperature may exceed 135°C during the dry cycle then delete the dry cycle.

- Autoclave sterilization is recommended for the product. The validity of other sterilization methods is not confirmed.
- Do not touch the product immediately after autoclaving as it will be very hot and must remain in a sterile condition.

## NOTICE


- NSK recommends Class B sterilizers as stated in EN13060.

## 8. Replacing the Cartridge (FX23/FX23m/FX25/FX25m)

- 1) Detach the shank by turning it counter clockwise and remove the head from the shank. (Fig.6)
- 2) Insert the wrench firmly into the rear of the handpiece head until it stops, as shown in Fig.A-1.
- 3) Tightly hold the wrench as illustrated in Fig.B then pull the drive shaft straight out in the direction of the arrow. (Fig.B)
- 4) Locate the correct wrench tool for the head cap then turn the wrench counter clockwise to loosen the cap. Remove the cap. (Fig.7)

## ⚠ CAUTION

- Be sure to insert the wrench straight into the handpiece head. Do not insert the wrench at an angle as shown in Fig.A-2.
- When removing the drive shaft, tightly hold the wrench as directed in Fig.B.

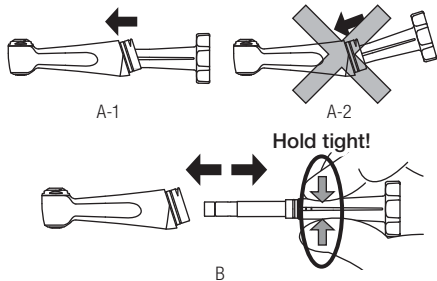- 5) Use the bur to gently lever the entire cartridge out from the head. (Fig.8)
  - 6) Clean the head interior with NSK PANA SPRAY Plus.
  - 7) Wipe the NSK PANA SPRAY Plus oil over the head interior.
  - 8) Insert the new cartridge into the head by aligning the pin on the cartridge with the slot on the head. (Fig.9)
  - 9) Tighten the head cap with the correct wrench.
  - 10) Insert the drive shaft firmly into the handpiece head until it clicks. If the click is not heard, insert it again.
  - 11) Reattach the shank.
- \*Refer to Spare Parts List to identify the correct cartridge.

## CAUTION

- After removing, do not hold the shank upside down as it could cause the internal parts to drop.
- Before turning the head cap, make sure the wrench is correctly aligned with the cap. Do not turn the wrench at an angle as abrasion of the wrench may occur.
- Secure the shank firmly to the handpiece head so that there is

no gap between them. A loose connection could make the head unstable, resulting in an accident during use.

- Use only a genuine NSK cartridge.
- If a non genuine cartridge is used NSK cannot guarantee performance and the handpiece warranty would become invalid.
- Always first finger tighten the head cap then secure firmly with the wrench.
- NSK never recommends the disassembly and repair of any NSK cartridge. There is NO EXCEPTION. In such a case a handpiece may perform abnormally (abnormal noise or abnormal vibration). Damage, failure or accidents are outside of our guarantee.

## 9. Periodical Maintenance Checks

Perform periodical maintenance checks every three months, referring to the check sheet below. If any abnormalities are found, contact your Authorized NSK Dealer.

Points to check	Details
Head cap is loose	Tighten firmly using the correct head cap wrench.
Rotation	Rotate the handpiece and check for abnormalities such as abnormal rotation, vibration, noise, and overheating.

## 10. Symbol

 This product is Autoclavable up to Max.135°C.

 This product can be washed via Thermo Disinfector.

 Conforms to CE European Directive of "Medical equipment directive 93/42/EEC."

 Manufacturer.

 Authorized representative in the European community.

 Caution: U.S. Federal law restricts this device to sale by or on the order of a licensed physician.

## 11. Warranty

NSK products are warranted against manufacturing errors and defects in materials. NSK reserves the right to analyze and determine the cause of any problem. Warranty is voided should the product be not used correctly or for the intended purpose or has been tampered with by unqualified personnel or has had non NSK parts installed. Replacement parts are available for seven years beyond discontinuation of the model.

## 12. Option Parts List

Model	Order Code	Compatible Product
FX Spray Nozzle	C1052702	FX23/FX23m/FX25/FX25m

## 13. Spare Parts List

Model	Order Code	Compatible Product	Remarks
FX23-03	C1056700	FX23/FX23m	Cartridge
FX25-03	C1052700	FX25/FX25m	Cartridge
FX Wrench	C1052701	FX23/FX23m/FX25/FX25m	

## 14. Disposing product

In order to avoid the health risks of operators handling the disposal of medical equipment, as well as the risks of environmental contamination caused thereof, a surgeon or a dentist is required to confirm the equipment is sterile. Ask specialist firms who are licensed to dispose of specially controlled industrial wastes, to dispose the product for you.

## 15. Specifications

Model	FX15	FX15m	FX23	FX23m	FX25	FX25m	FX75	FX75m
Max. Rotation Speed (Motor)	40,000 min <sup>-1</sup>		30,000 min <sup>-1</sup>		40,000 min <sup>-1</sup>		20,000 min <sup>-1</sup>	
Max. Rotation Speed (Handpiece)	10,000 min <sup>-1</sup>		30,000 min <sup>-1</sup>		40,000 min <sup>-1</sup>		1,250 min <sup>-1</sup>	
Gear Ratio	4:1 Reduction		1:1 Direct Drive		1:1 Direct Drive		16:1 Reduction	
Bur Type	ISO 1797-1 Type1 ø2.35mm CA Bur						ISO 1797-1 Type1 ø2.35mm Engine File	
Chucking Length	11.5mm						9.4mm	
Max. Bur Length	22.5mm						21.0mm	
Max. Working Part Diameter	ø4.0mm						ø1.45mm	
File Motion	-						360°Rotation	
Use Environment	Temperature: 0-40°C (No Condensation), Humidity: 30-75%, Atmospheric Pressure: 700-1,060hPa							
Transportation and Store Environment	Temperature: -10-50°C, Humidity: 10-85%, Atmospheric Pressure: 500-1,060hPa							
Thermo Disinfector	-	○	-	○	-	○	-	○

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2013.02.20 05 ©
**NSK United Kingdom Ltd**
**NSK America Corp**
**NSK Oceania Pty Ltd**
**NSK Asia Pte Ltd**